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Model: B-DBP452


- Please read the product "Instruction Manual" carefully before using, purchasing and using it under the guidance of the doctor.
- Please refer to the "Instruction Manual" for the taboo contents or

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Motherboard Diagram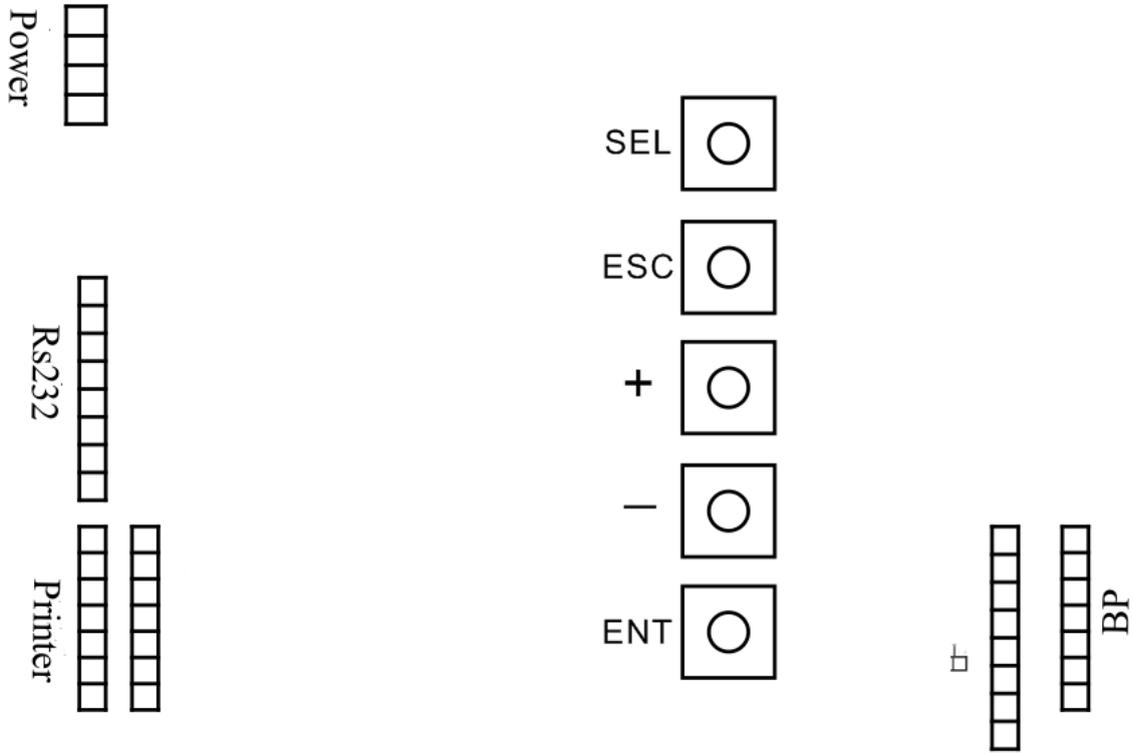


Operation


- Startup:** Connect the power cord, turn on the power, and the display will show the initialization information
- Voice:** Automatically report measurement results (can be set)
- Print:** Automatically print the measurement results after measurement (can be set).

Steps


Step 1: Insert your upper arm into the blood pressure monitor arm cup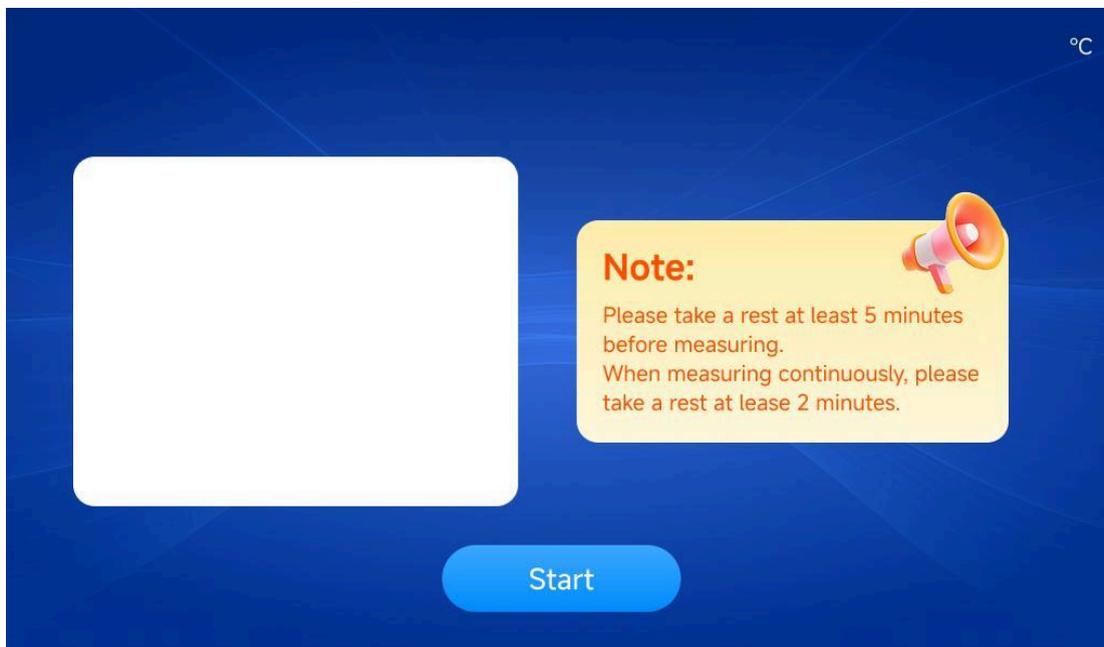


Step 2: When ready, press the "Start" button to measure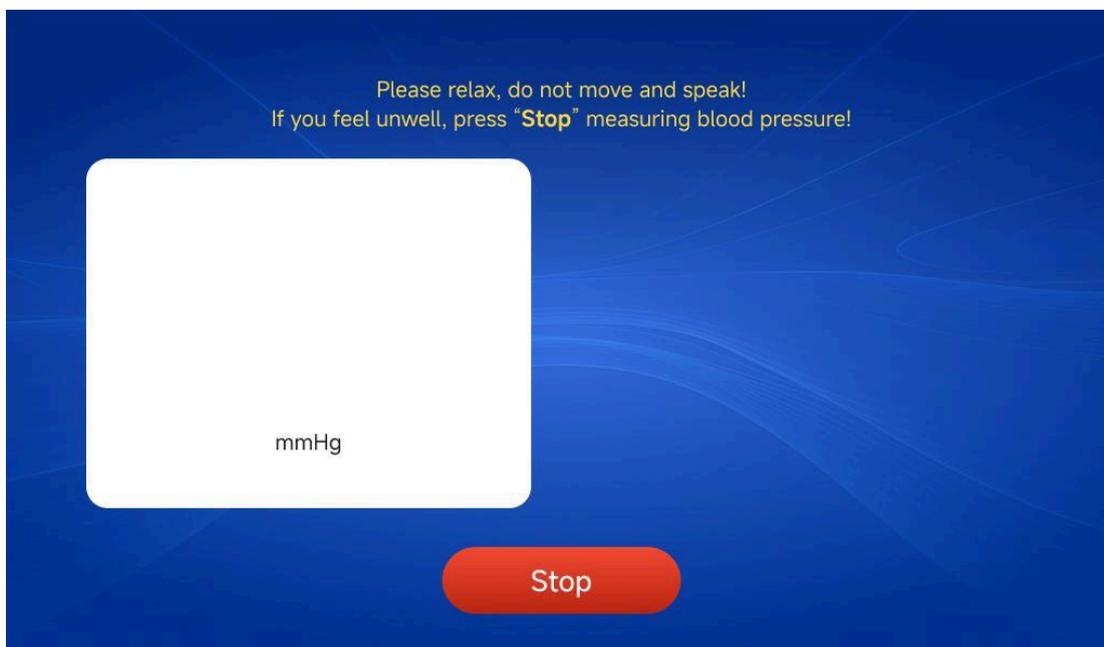

Step 3: Get the printed results (after the measurement is completed, the machine automatically prints the measurement results).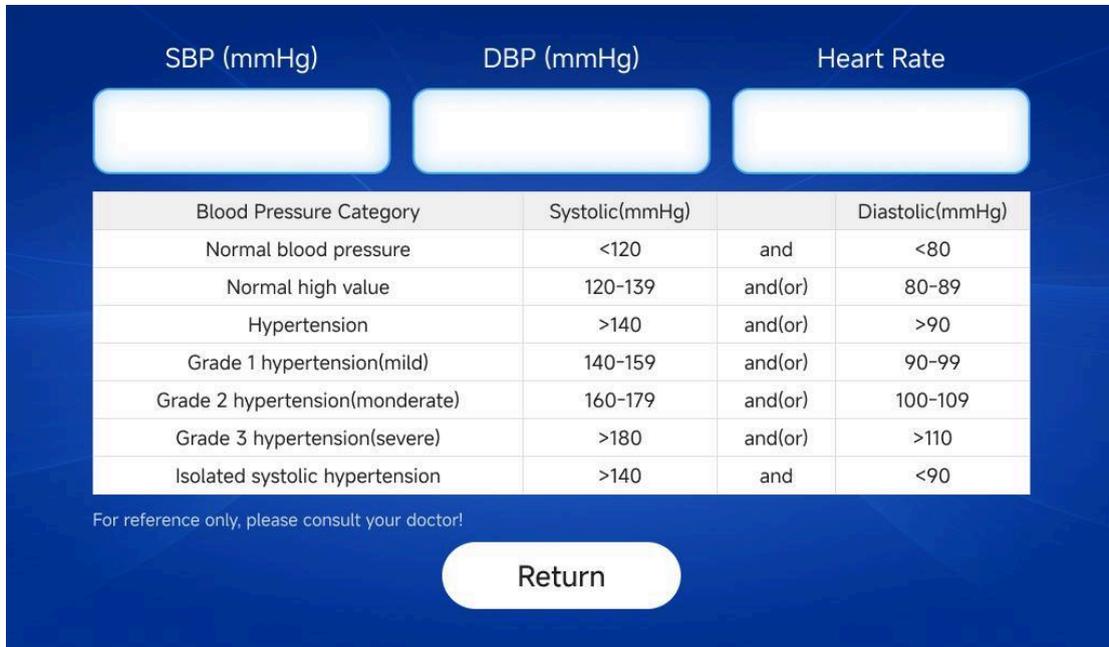

Note: If you feel uncomfortable during the measurement, you can press the "Stop" button to stop measuring blood pressure. If the blood pressure measurement fails, you can repeat the measurement by Start button. If you want to give up the measurement, press the button and the machine will return to standby mode.

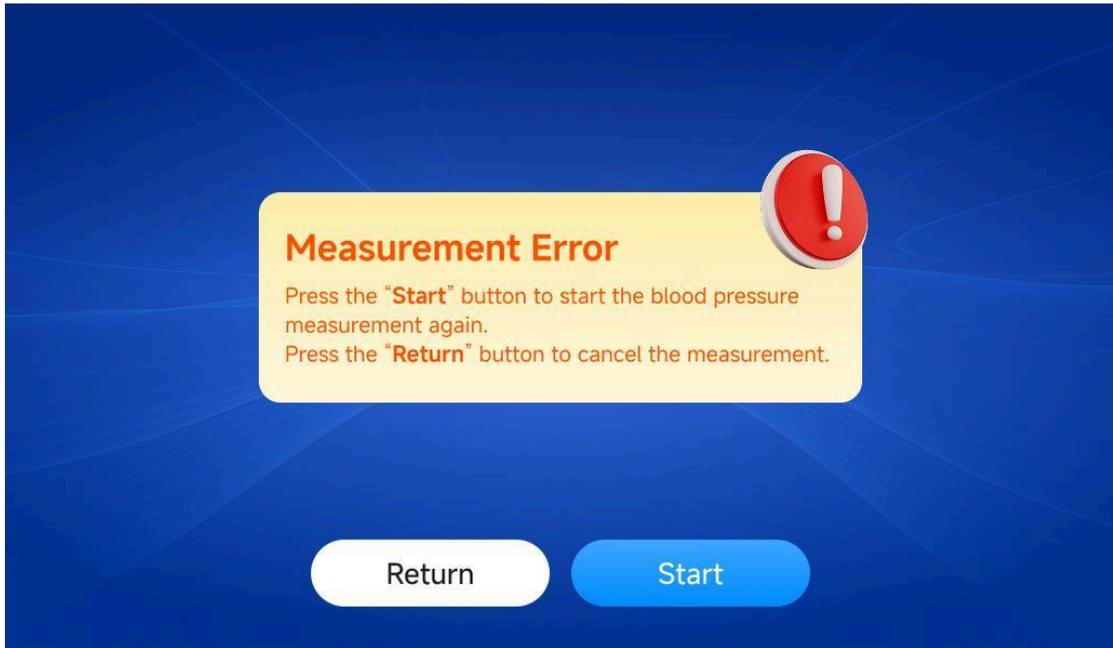
Function Setting

In standby mode, press the **ENT** key to enter the menu

Date and time

Press **Date/Time** Button and input the date and Time information on the Input box as the picture. Press **Enter** to save the adjustment results and exit. Press **Cancel** do not save the adjustment results and exit.

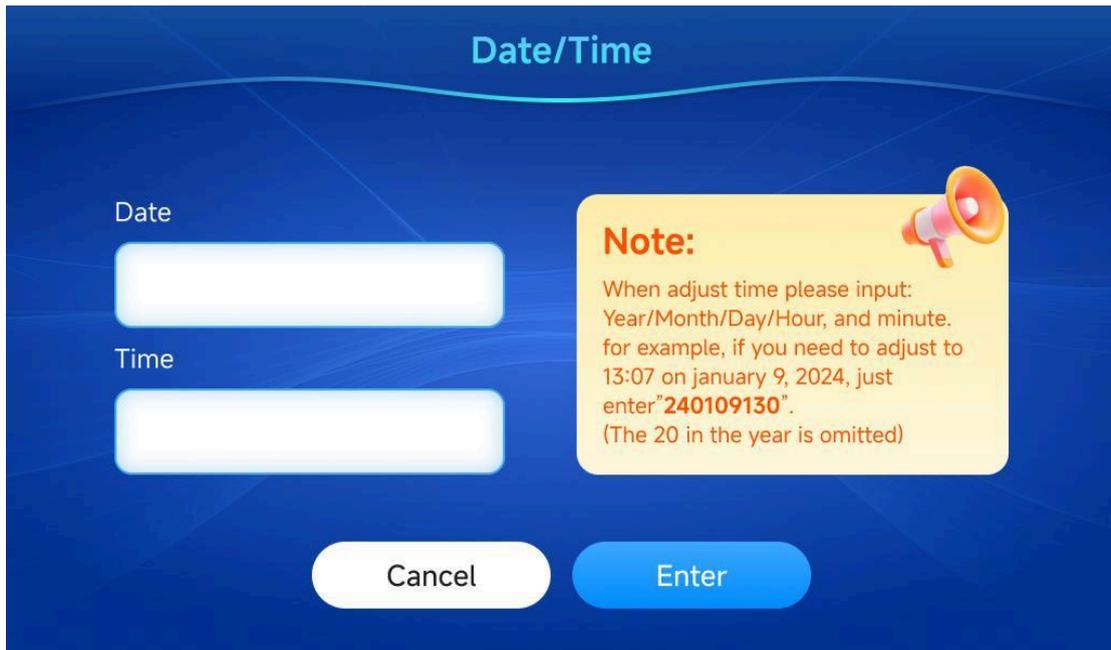The image shows a 'Date/Time' adjustment screen with a dark blue background. At the top center, the title 'Date/Time' is displayed in light blue. On the left side, there are two white input fields: the top one is labeled 'Date' and the bottom one is labeled 'Time'. To the right of these fields is a yellow callout box with a megaphone icon and the text: 'Note: When adjust time please input: Year/Month/Day/Hour, and minute. for example, if you need to adjust to 13:07 on January 9, 2024, just enter "240109130". (The 20 in the year is omitted)'. At the bottom of the screen, there are two buttons: a white 'Cancel' button and a blue 'Enter' button.

Print

Click the "Print" button to enter the settings. You can set the print function and the header and footer content. Click the header/footer to turn the header/footer function on or off.

The image shows a 'Print' settings screen with a dark blue background. At the top center, the title 'Print' is displayed in light blue. In the top right corner, there is a 'Test Print' button with a printer icon. The main area contains four blue buttons arranged in a 2x2 grid. The top-left button is labeled 'Print' with a checked radio button icon. The top-right button is labeled 'Header' with a gear icon. The bottom-left button is labeled 'No Print' with an unchecked radio button icon. The bottom-right button is labeled 'Footer' with a gear icon. At the bottom of the screen, there are two buttons: a white 'Cancel' button and a blue 'Enter' button.

Click the corresponding settings button to enter the content settings, then Press ENTER or Cancel button the save or cancel the setting

Header/Footer

Cancel Enter

Voice

Click the **Voice** button to enter the settings. You can set the voice status and adjust the volume. Adjust the measurement preparation time and the measurement result display time. Click the input box to enter the time in seconds. Click the **Enter** button to save the settings.

Voice

Play Video Result Voice Prompts Broadcast When Standby

Voice Cycle [] S
Prepare Time [] S
Display Result [] S

— Volume +

Cancel Enter

Count

Press **Count** to check the total measurement Times; Press **Clear** to clear the measurement times; Press **Print** to print and clear the measurement times; Press **Cancel** to cancel all settings.

The Current Number

The Total Number

Prompt: Click the print key to automatically clear the current number.

Cancel Print

System Set

Input the Content information (English Words only), then press **Enter** button, these contents will display on the standby screen.

System Set

Company

Cancel Enter

Reset

Click the **Reset** button to enter the settings interface. Click the **Enter** button to restore the factory settings.


Introduction

Caution

- This device is intended for use by adults. Do not use this device on neonates or infants. Do not use on children unless under the guidance of a medical professional.
- This device is intended for non-invasive measurement and monitoring of blood pressure.
- Do not use this device for diagnosis or treatment of any disease or health condition. Consult your physician if you have or suspect any medical condition. Do not change your medications without the advice of your physician or healthcare professional.
- This device is not suitable for patients with implanted electrical devices, such as defibrillators or cardiac pacemakers.
- If you are taking medication, consult your physician to determine the appropriate timing to measure your blood pressure.
- Do not use the device if it is damaged or malfunctioning.

Host description

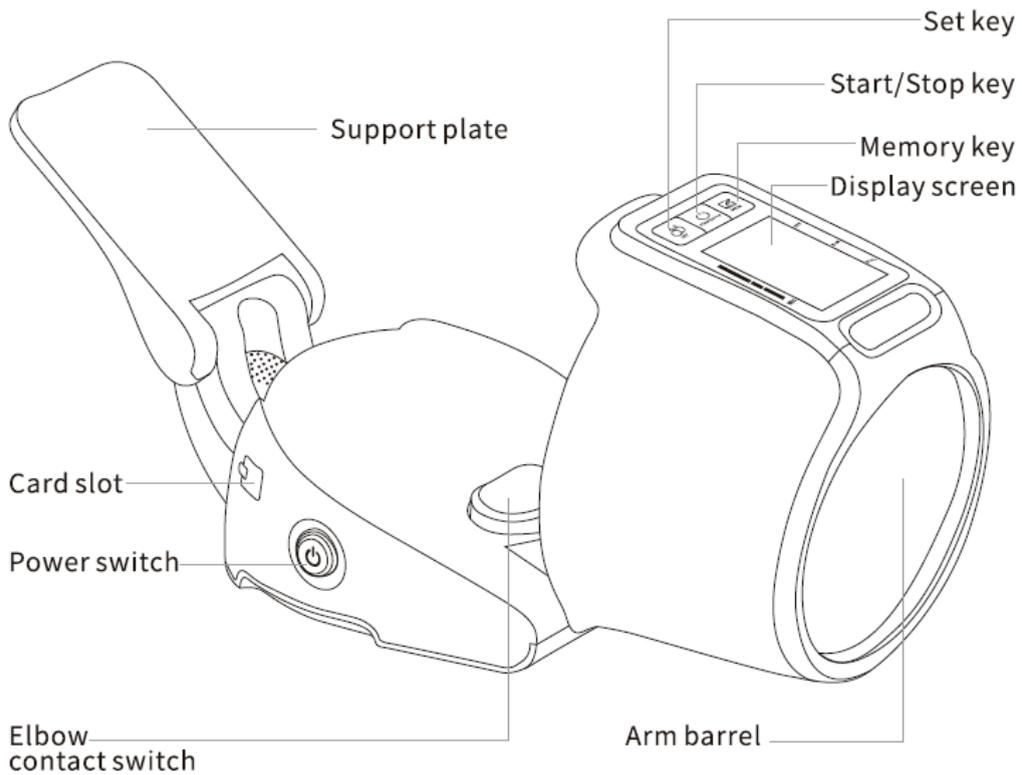Function description of each key

key: Short press this key in the off state to perform user settings. In the

off-state press and hold for 3 seconds to enter the device setting state, and you can set the voice on and off, blood pressure unit in turnout.


key: Press this key in the off state to view the blood pressure measurement value stored in the device.

key: Press this key in the off state, the device enters the process of automatic blood pressure measurement.

LCD display description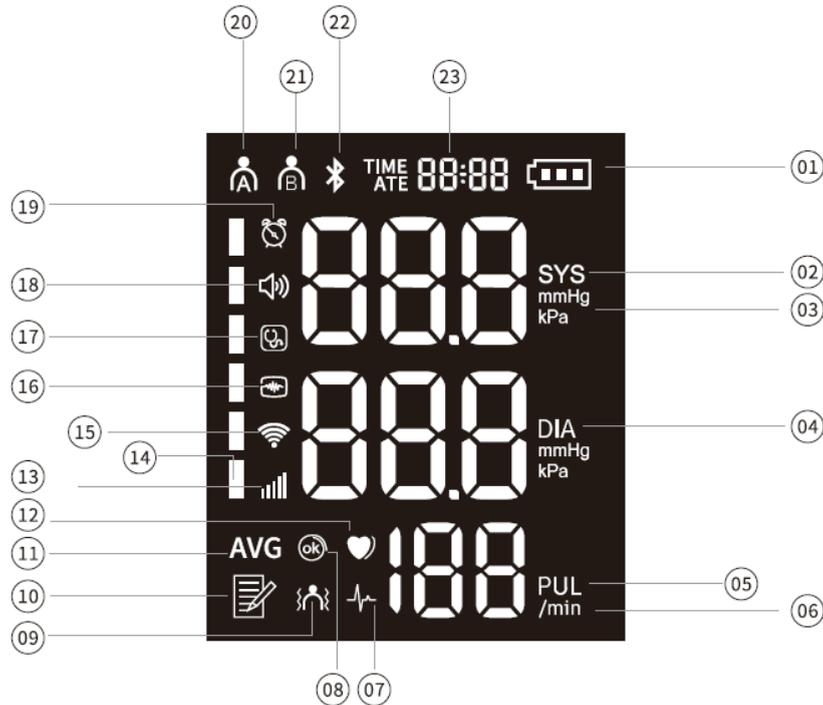

1. Battery icon
2. Systolic value display
3. Blood pressure unit
4. Diastolic value display
5. Pulse value display
6. Pulse value unit
7. Irregular heartbeat icon
8. Arm barrel self-test icon
9. Anti-shake
10. Number of memory sets
11. Average value
12. Heartbeat icon
13. GPRS icon
14. Blood pressure indicator bar
15. Wifi icon
16. Oscillography
17. Auscultatory method
18. Voice icon
19. Alarm icon
20. User A
21. User B
22. Bluetooth icon
23. Time icon

Specifications


Product name:	Electronic Upper Arm Blood Pressure Monitor
Measurement method:	Oscillography + Auscultatory method

Measuring range:	Pressure: 0 mmHg ~ 295 mmHg (0.0 kPa ~ 39.33 kPa) Pulse rate: (40 ~ 199) beats/min
Allowable error:	Pressure: ± 3 mmHg (± 0.4 kPa) Pulse rate: $\pm 5\%$
Resolution:	1 mmHg / 0.1 kPa
Power adapter voltage:	Input: 100-240V ~ 50/60Hz 1A-0.3A Output: DC6V/4A
Normal working environment:	Temperature: 5°C ~ 40°C Relative humidity: 15% ~ 80% Air pressure: 80 kPa ~ 105 kPa
Transportation and storage environment:	Temperature: -20°C ~ + 55°C Relative humidity: 15% ~ 93% Air pressure: 70 kPa ~ 106 kPa
Memory number:	2*99 sets
Applicable arm circumference:	24 cm - 42 cm
Product size:	Length 189 mm * width 263 mm * height 233 mm
Net weight:	About 1.5 kg

How to use


1. User Selection

In the shutdown state, short press “” key once to enter the user Settings state, the icon of user A or user B is displayed On, Short press the “” key to select user A or user B. Short Press the “” key to confirm the shutdown.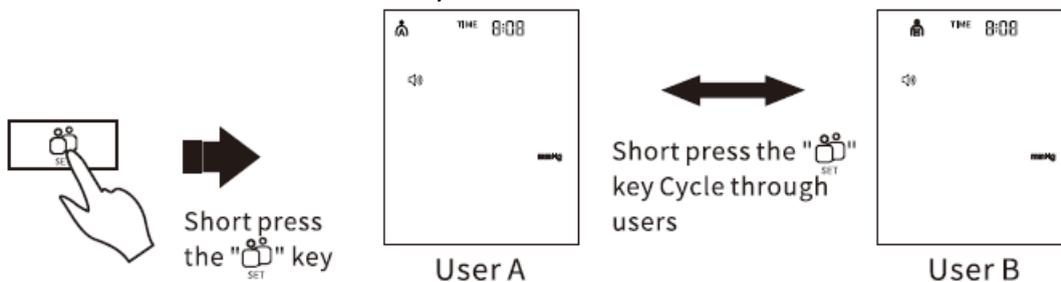


2. Time Setting

In the shutdown state, hold down the “” key for 3 seconds to automatically enter the time setting mode, the time icon indicates "year" the numeric character will blink, at this time, you can short press the “” key to select the required year, and short press the “” key to confirm it, set the month, date, hour, and minute in this way. (if the time is not set, the default is the machine factory time)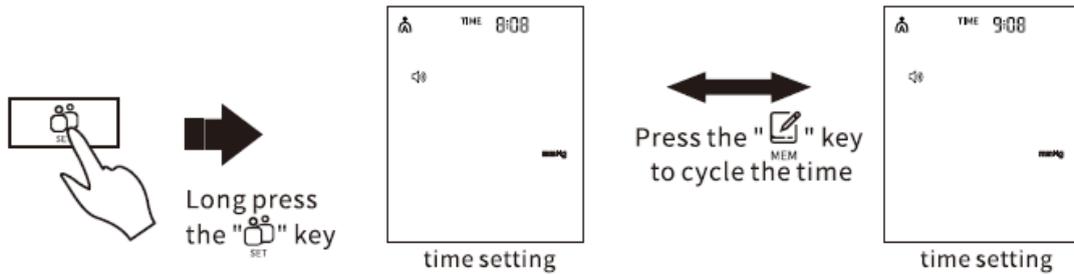

3. Voice Setting


After the time setting is complete, the system automatically enters the voice setting mode, and the screen displays "SP 50", "50" will flash at the bottom, short press the "MEM" button repeatedly to switch the volume gear, press once to increase the volume by one step, the volume level is displayed as SP 00, SP 25, 50, 50, 75, and SP 100 in sequence, select a volume level after that, short press the short "SET" key to confirm.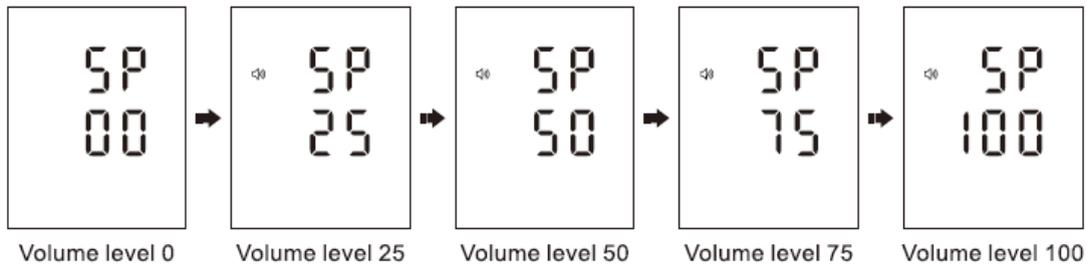

4. Blood pressure value unit setting

After the voice setting is complete, the screen will automatically enter the blood pressure unit setting mode, "PA" will be displayed, and "OFF" will blink at the bottom, short press "MEM" to switch between "OFF" and "ON", select "OFF" to select mmHg


as the unit, select "ON" to select kPa as the unit, short press "MEM" select the blood pressure unit. Press the Set key to confirm.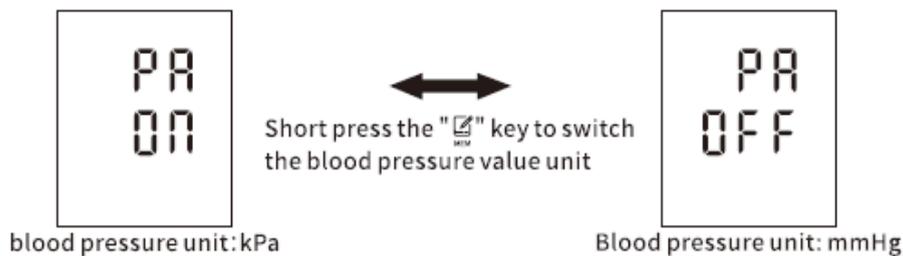

5. Auscultatory method mode Settings

After the blood pressure unit is set, it will automatically enter the auscultatory method mode setting, the screen will display "AS", while the "ON" below will blink, the icon of "👂" will also blink, short press the "MEM" key to switch "ON" and "OFF", select "ON" to enable auscultatory method mode, select "OFF" to disable

auscultatory method mode, short press the “MEM” key to select whether to enable auscultatory method mode, press the “SET” key to confirm (auscultatory method mode is enabled by default).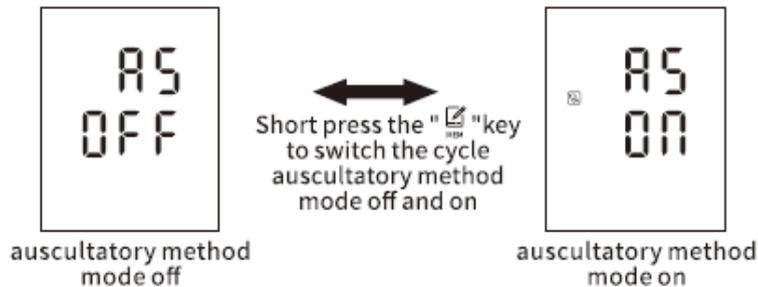

6. Bluetooth Connection

1. Press the set key or memory key to light up the screen;
2. The Bluetooth icon blinks when the Bluetooth is not connected.
3. After the Bluetooth connection is successful, the Bluetooth icon is always on;

Start measuring

1. After sitting in the correct posture, insert the arm into the arm tube and adjust the position of the arm and elbow.
2. Press the “START/STOP” key, and the blood pressure monitor will prompt you to start measuring now.

Warm tips:


1. If you feel uncomfortable during the measurement process, just press the “STOP” key to vent the air and enter the emergency stop mode.
2. When the auscultatory method signal is abnormal, the “” icon will not appear on the display interface.
3. When the heartbeat is irregular, an irregular heartbeat icon will appear in the lower left corner of the pulse on the display interface.
4. Ask a professional to explain the measured blood pressure value

Correct measurement posture

Step1

Straighten your body, maintain a correct sitting position, and breathe deeply for 5-6 times before the measurement, then return to nature breathe, relax the shoulders and arms to relax the whole body;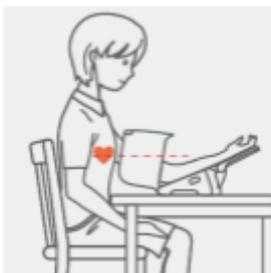

Step2

When measuring, the middle of the arm barrel is consistent with the height of the heart, and during the measurement,


the hand does not force do not speak, the body remains still until the measurement is completed;

Tips

Sit still for 5 minutes before the measurement, and then take the measurement with the correct posture (guaranteed mood quiet, stable), when multiple measurements were taken, when two measurements were taken next to each other interval of at least 2-3 minutes or longer;

Measurement result prompt

If any of the following situations occurs, the display will display the measurement result prompt, as shown in the table below

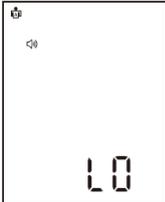
Measurement result prompt	Prompt reason
	Pulse below 40
	Pulse over 199

Storage measurement usage

The blood pressure monitor stores 2*99 sets of memory values. after each measurement, the blood pressure monitor will automatically store the measurement data. the memory capacity is fully loaded, and the old measurement data will be overwritten by the new data.

View memory

In the off state, short press “”, the machine will display the average value of the previous three blood pressure measurements, and then short press “” then the blood pressure value measured for the first time will be displayed, and the second short press will be displayed for the second blood pressure measurement value, and so on. See the diagram below:

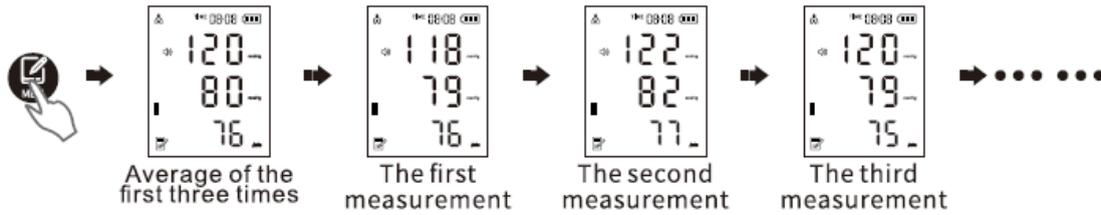Delete memory in the off state

Press the “” key once, and after the memory value appears, press and hold the  key until all the data disappears, that is, delete all the memory values under the current user in the machine.

Notice

The blood pressure monitor cannot delete part of the measurement results. If you want to delete the measurement values stored in the blood pressure monitor, all the measurement values stored in the blood pressure monitor will be deleted (It is recommended that you make a blood pressure record before deleting the measurement values for future reference).

Measurement error prompt

If any of the following situations occurs, the display will display an error message, as shown in the table below

Error message	Possible cause of error	Solution
Err 1	Sensor anomaly	1. Squeeze the arm cylinder before measuring to empty the air inside the arm cylinder
Err 2	No pulse signal or abnormal pulse signal was detected	1. Insert the arm cylinder correctly according to the position indicated in the instruction manual, and place the elbow correctly 2. Keep quiet and do not move during the measurement 3. Don't take measurements in thick clothes
Err 6	Abnormal pressure interferes with the cylinder during measurement	1. Keep quiet and do not move during the measurement.
Err P	Inflation anomaly	1. Check whether the air circuit is open 2. Check whether the arm barrel is fitted too tight or too loose 3. Check whether the trachea is blocked.

Err H1	Pressure over limit	1. Measure according to the required posture, and do not move during measurement.
--------	---------------------	---

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Product Owner: Lifebox Asia Company Limited